DAVIS

QUEAZ-AWAY™

TRAVELERS' WRISTBANDS


Queaz-Away™ Travelers' Wristbands

Queaz-Away™, the "tummy tamer," is a non-drug aid for people on the go. The unpleasant effects of winding roads, rough seas, bumpy flights, rocking trains, and spinning Ferris wheels can ruin a trip or outing. Queaz-Away wristbands exert gentle pressure on points of each wrist, easing the miserable effects of travel. Made of tough, long-lasting, washable materials. Safe and easy to use. Simply slip onto wrist, position, and enjoy your travel. No complicating buckles or adjustments. May be worn repeatedly or continuously, even while sleeping to prevent the onset of discomfort. Used successfully throughout the world by yachtsmen, pilots, divers, physicians and hospitals. One size fits all, adults and children. Comes with handy carrying case.

MUST BE WORN PROPERLY TO BE EFFECTIVE

The bands must be worn on both wrists. Position the button as follows (see illustration): Starting at the first wrist crease (A), use your three middle fingers (B) to measure to a point on the underside of each wrist (C). Position the button downward over this point, between the two central (flexor) tendons (D).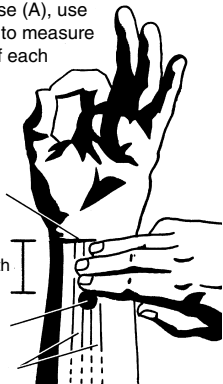

(A) First wrist crease

(B) Three fingers' width

(C) Button placement, face down

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OTHER DAVIS PRODUCTS FOR PEOPLE ON THE GO:

- A wide selection of Marine Gear for sailboats and powerboats
- A full line of Weather Instruments for your home, boat, or cabin

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3465 Diablo Ave., Hayward, CA 94545, U.S.A.